PERIO-FLUSH

rev. 2, 18.05.2023

PL Roztwór do płukania kieszonek przyzębnych w dozowniku strzykawkowym
EN Solution for the irrigation of periodontal pockets in a syringe dispenser
FR Solution pour l'irrigation des poches parodontales dans une seringue
DE Lösung zur Spülung von Zahnfleischtaschen in einem Spritzenspender
ES Solución para la irrigación de las bolsas periodontales en jeringa dosificadora
IT Soluzione per l'irrigazione delle tasche parodontali in un dispenser a siringa
BG Разтвор за промиване на пародонтални джобове в диспенсърна спринцовка
CS Roztok pro vyláplav parodontálních důškových rotnocí injekční dvákovka
RO Soluție pentru irigații parodontale într-o seringă dozatoare
PT Solução para a irrigação de bolsas periodontais em dispensador composto por uma seringa

PL	EN
FR	DE
ES	IT
BG	CS
RO	PT

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PERIO-FLUSH – Solution for the irrigation of periodontal pockets in a syringe dispenser
[MD]Class I medical device

PERIO-FLUSH is a ready-to-use liquid solution for the irrigation and rinsing of periodontal pockets. It can be used to support non-surgical treatment of periodontal pockets. It effectively removes solid impurities from the pocket. It does not disturb the regeneration of connective tissue attachment.

MECHANISM OF ACTION
PERIO-FLUSH contains a solution of mineral salts with the addition of a quaternary ammonium compound (domiphen bromide). The main action of PERIO-FLUSH consists in mechanical removal of inflammatory content from the periodontal pocket.

INDICATIONS FOR USE

- Inflammation of periodontal pockets, in particular:
 - around class II fillings or areas with no contact points,
 - around teeth with difficult eruption, especially wisdom teeth,
 - around teeth intended for extraction,
 - around crowns,
 - around teeth supporting denture clasps, clips or removable braces,
 - around fixed orthodontic appliances.
- Soft tissue inflammation under restorations such as bridges;
- Soft tissue inflammation under dentures and removable braces;
- Rinsing the pockets after scaling.

GENERAL HANDLING INSTRUCTION

If possible, bring PERIO-FLUSH to body temperature to minimise patient discomfort during application. Unscrew the cap and mount the disposable applicator by screwing it on the luer-lock of the syringe. Make sure the dosing system is working correctly by gently pressing on the plunger above from the patient's mouth until a drop of the product can be seen. If it is impossible to extrude the product, replace the applicator with a new one. If the process is successfully completed, you can start working with the patient. To do that, gently insert the applicator down to the bottom of the periodontal pocket. Upon reaching the bottom, retract the applicator to 1-2 mm from the bottom of the pocket and start irrigating by slowly pressing on the syringe plunger. Below you will find detailed instructions on how to proceed in different clinical cases. Continue until the liquid flowing out of the pocket is clear. After the procedure is finished, instruct the patient to spit out any remaining product and rinse their mouth with mouthwash, preferably MUSEON solution. Detach and dispose of the use applicator and the empty syringe as instructed.

GENERAL HANDLING INSTRUCTION
If possible, bring PERIO-FLUSH to body temperature to minimise patient discomfort during application. Unscrew the cap and mount the disposable applicator by screwing it on the luer-lock of the syringe. Make sure the dosing system is working correctly by gently pressing on the plunger above from the patient's mouth until a drop of the product can be seen. If it is impossible to extrude the product, replace the applicator with a new one. If the process is successfully completed, you can start working with the patient. To do that, gently insert the applicator down to the bottom of the periodontal pocket. Upon reaching the bottom, retract the applicator to 1-2 mm from the bottom of the pocket and start irrigating by slowly pressing on the syringe plunger. Below you will find detailed instructions on how to proceed in different clinical cases. Continue until the liquid flowing out of the pocket is clear. After the procedure is finished, instruct the patient to spit out any remaining product and rinse their mouth with mouthwash, preferably MUSEON solution. Detach and dispose of the use applicator and the empty syringe as instructed.

INSTRUCTIONS FOR USE

INFLAMMATION OF PERIODONTAL POCKETS:

- around class II fillings or areas with no contact points**
Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove solid plaque, taking care not to damage the root cementum. While irrigating, move the applicator around all the tooth roots, keeping the tip close to the bottom of the pocket. Go round each tooth and root separately, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.

around teeth with difficult eruption, especially wisdom teeth
Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument. While irrigating, move the applicator around the tooth, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.

around teeth supporting denture clasps, clips or removable braces
Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument. While irrigating, move the applicator around the tooth, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.

around crowns
Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument, also under the crown. While irrigating, move the applicator around the tooth and under the crown, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue rinsing as long as there is any inflammatory content or solid impurities flowing out of the pocket or from under the crown. While rinsing as long as there is any inflammatory content or solid impurities flowing out of the pocket, make sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket, and until the site where extraction instruments are to be inserted is clean.

around crowns
Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument, also under the crown. While irrigating, move the applicator around the tooth and under the crown, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue rinsing as long as there is any inflammatory content or solid impurities flowing out of the pocket or from under the crown. While rinsing as long as there is any inflammatory content or solid impurities flowing out of the pocket, make sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.

around fixed orthodontic appliances
Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove solid plaque, taking care not to damage the root cementum. While irrigating, move the bracket to a safe distance from the gingiva (an additional 2 mm is usually enough). While irrigating, move the applicator around the tooth, keeping the tip close to the bottom of the pocket. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.

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