Zhermack S.p.A. Via Bonaventuro, 100 - 46021 Bari (Bari) (Italy)  
T +39 0424 99 61 11 www.zhermack.it

**100 wipes**

dermatologically tested

specific for surfaces

**zeta 3 wipes**

POP-UP

Dermol

**Efficacia**

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**EN - INDICATION OF USE:** Wipes designed to clean and disinfect non-invasive and non-submersible medical devices (small medical and dental devices).

**INSTRUCTIONS FOR USE:** Ready-to-use wipes. Single use wipes. When using the sachets the first time:

- Open the winder, permanently remove the tamper-proofable.
- Thread the first wipe through the hole of the winder with clean hands and then close the winder.
- Use the wipe on the equipment.
- Leave on for 30 seconds to 15 minutes according to the antibacterial activity required. Leave to dry in the open air. Do not rinse.
- Close the winder carefully after each use to prevent the wipes from drying out.

To be used within three months after opening.

**PRECAUTIONS:** Keep out of reach of children. Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents/container to an approved waste disposal plant. Wear protective gloves. If in eyes: Rinse cautiously with water for several minutes. Safety data sheet available on request. Never reuse wipes, they may be less effective and there is a risk of croo contamination. To be used within 3 months after opening. Dermatologically-tested for skin tolerance.

**CONTAINS IN PARTICULAR:** Less than 5% non-ionic surfactants perfumes (including limonene), disinfectants : Ethanol (CAS 64-17-5), Alkyl (C12-16) dimethylbenzylammonium chloride (CAS 68424-85-1), Propan-2-ol (CAS 67-63-0).

## SPECTRUM OF ACTION

Effectiveness*		Effective period under clean conditions
<b>General treatment</b>		
Bactericidal	EN 13727** EN 14561** EN 14561: MRSA	5 min
Fungicidal	EN 14562 (Aspergillus fumigatus)	5 min
Yeasticidal	EN 13624, EN 14562 (Candida albicans)	5 min
Tuberculocidal	EN 14348, EN 14563 (Mycobacterium terrae)	15 min
Virucidal	EN 14476**(HBV, HCV, Adenovirus, Coronavirus, Norovirus, VRS, H1N1, HSV1), EN 14476 (Poliovirus) EN 14476 (Rotavirus)	1 min 15 min
<b>Additional uses</b>		
Bactericidal	EN 1276 EN 13697	30 s 5 min
Yeasticidal	EN 1650**, EN 13697 (Candida albicans)	5 min

\*Properties of the impregnation solution. \*\*In dirty conditions.

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